


Eagle Eye Anesthesia

11233 St John's Industrial Parkway S. Jacksonville, FL 32246 800-760-6976 www.eagleeyeanesthesia.com


Installation Of Vaporizer One-Way Check Valve



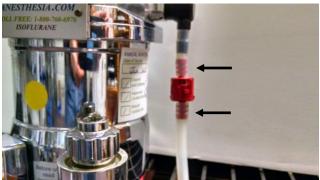

Remove the current opaque silicon hose from the vaporizer outlet adapter which is located on the right (outlet) side of the vaporizer if you are facing the front of the vaporizer.

Cut the hose you removed approximately 2 inches from the end. Prior to installing the valve locate two small arrows on the body of the valve. When installing the valve these arrows must be pointing down. Install the top barbed portion of the oneway valve into the short hose by sliding it into the short hose. Install the other barbed end of the one-way valve into the long hose by sliding it into the long hose. Reattach the hose with the oneway valve back onto the vaporizer adapter on the vaporizer outlet.

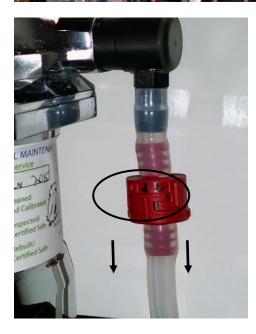

The arrows MUST point downward in the direction of gas flow.